

ManaEZ BOOT AIR

Plastic Camboot with Pneumatic Pump

INSTRUCTIONS FOR USE



Features

- **Rocker bottom to simulate natural gait**
- **Pneumatic Pump and Tall Design**
- **Large air bladder creates a larger surface for contact pressure to help control swelling and ensure a snug fit**



To put on the ManaEZ Boot Air:

1. Open the boot liner without removing it from the boot.
2. Place the boot flat on the floor and put the foot inside the boot. Make sure the heel is all the way back to the back of the boot.
3. Secure the lining to the leg by overlapping the liner on the top of the foot and at the front of the lower leg.
4. Secure the Velcro straps beginning with the straps closest to the toes.

To Control the Air Pressure:

1. To put air inside the boot, turn the valve clockwise until secure, and then squeeze the bulb until you feel adequate pressure.
2. To remove air from the boot, simply turn the valve counterclockwise.

Product Name	Men's Shoe Size	Women's Shoe Size	Part Number
ManaEZ Boot Air Extra Small	Up to 5.5	Up to 4	EZBA01XS
ManaEZ Boot Air Small	6 - 8	4 - 7	EZBA01S
ManaEZ Boot Air Medium	8 - 11	7 - 10	EZBA01M
ManaEZ Boot Air Large	11 - 13	10 - 12	EZBA01L
ManaEZ Boot Air XLarge	13 +	12 +	EZBA01XL



MANAMED™

ManaMed Inc.
5240 W Charleston Blvd.
Las Vegas NV 89146
United States
Ph: 888-508-0712
www.ManaMed.com

©2020 ManaMed Inc. All rights reserved.
ManaMed, the ManaMed logo, EZ Boot Air logo,
are trademarks of ManaMed Inc.
Registered in the U.S.