

## INSTRUCTIONS FOR USE

# Mana **EZ** TENDON STRAP Universal

BEFORE USING THIS DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

### INTENDED USE:

Patellar tendonitis  
Patella tracking disorders  
Osgood-Schlatter disease  
Patellofemoral knee pain

### WARNINGS & PRECAUTIONS:

If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

### CLEANING INSTRUCTIONS:

1. Clean material with water and a mild cleanser.
2. Lay flat and air dry ONLY.

### WARRANTY:


Manamed guarantees all products to be free from defects in materials and workmanship for a period of 6 months from date of purchase.

### INSTRUCTIONS FOR USE:

1. Open the brace by pulling apart the Velcro.



2. Wrap the brace around the front of the leg, just below the patella

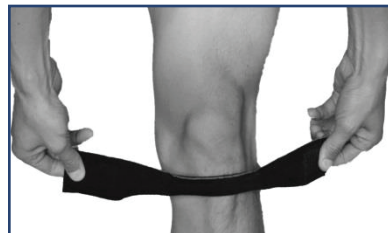

3. Attach the Velcro behind the knee, just below the patella. Walk around. Adjust tension as necessary.

