

Halfback50 ™



MANAMED™

Patents Pending.

Halfback50™




The Halfback50 Brace provides support for both spinal surgeries and conservative treatment of spinal injuries. The brace features universal sizing and multiple L code configurations. Halfback50 also features a patent-pending chord compression system and signature M shaped handle.

Sizing

Halfback 50 can be adjusted from 25" to 68".
Halfback 50 comes with a main panel and two adjustable belts with comfort pull rings.

Simply place the fullback on a table with the inside facing up.

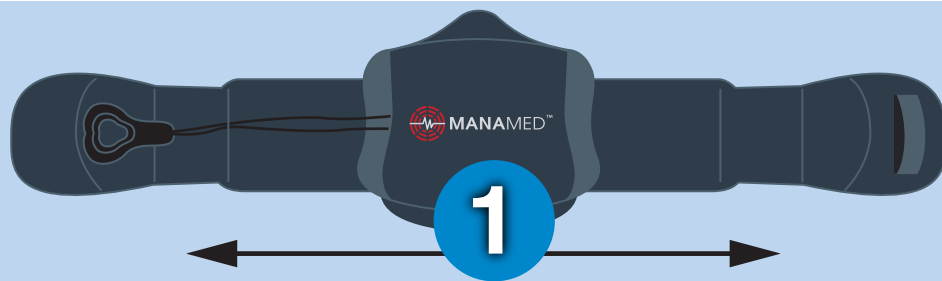

Step 1 Remove Velcro from belts and pull the tabs through to the creases, securing the sizing flap with Velcro to your appropriate size.

Features and Benefits

- Universal size with Low-profile design
- Simple One Pull Compression System
- Three Location Gear System
- Multiple L Code Configurations

Indications for Use


- | | |
|-----------------------|-------------------|
| Low Back Pain | Laminectomy |
| Osteoporosis | Discectomy |
| Disc Degeneration | Spinal Stenosis |
| Disc Herniation | Spondylolisthesis |
| Post-Operative Fusion | |



Step 2 Put the brace around your waist and secure the front panel onto the brace so the Velcro secures the fit.



Step 3 Pull both comfort rings forward until you feel comfortable and then secure the rings to your brace by placing them on to the sides. The comfort rings control the compression of the brace and can be used to adjust the fit.

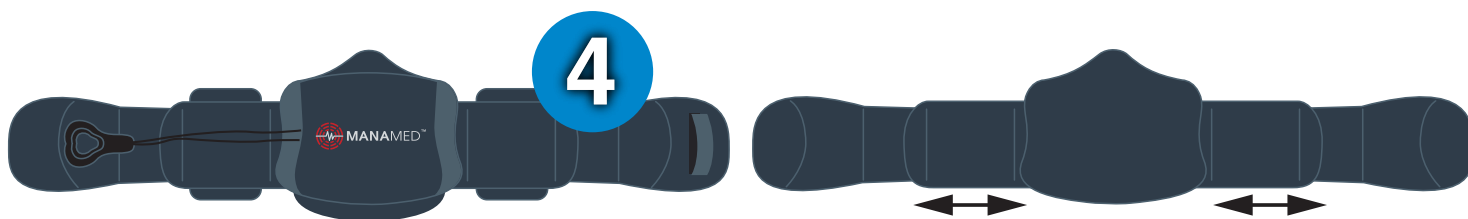

Step 4 (optional) Optional Lateral panels. If you are instructed to use the Lateral panels, secure them to inside of the brace by placing them next to the main panel, with the large curve resting next to the main panel on both sides.

Removing the brace Lift both comfort rings off the brace placing them to the sides and securing them so there is no tension on the brace. Then remove the Velcro connection on the front panel and safely remove the brace.

Order information

Part Number: HB050 Size: Universal

How to order

Email: orders@manamed.net, Fax: 888-508-0712, Call: 888-508-0712

5240 W Charleston Blvd.,
Las Vegas, NV 89146
www.ManaMed.com



MANAMED™