

ManaEZ BOOT Pediatric

Pediatric Plastic Camboot



INSTRUCTIONS FOR USE



The Kid Walking Boot is designed for the treatment of stable fractures and ankle sprains for Kids. It is used for early rehabilitation of a variety of post surgical and post trauma indications requiring fixed control.

- Rocker bottom and low profile heel facilitates a more natural gait pattern.
- Lightweight design includes anatomically shaped plastic uprights that help to facilitate a better fit.
- Easy to apply and adjust with a simple strapping and liner system.
- It fits right and left.

To put on the ManaEZ Boot Pediatric:

1. Open the boot liner without removing it from the boot.
2. Place the boot flat on the floor and put the foot inside the boot. Make sure the heel is all the way back to the back of the boot.
3. Secure the lining to the leg by overlapping the liner on the top of the foot and at the front of the lower leg.
4. Secure the Velcro straps beginning with the straps closest to the toes.

Product Name	Child's Foot Length	US Shoe Size	Part Number
ManaEZ Boot Pediatric Small	Up to 6"	Up to 8.5	EZBPED-S
ManaEZ Boot Pediatric Medium	6" - 7"	8.5 - 11.5	EZBPED-M
ManaEZ Boot Pediatric Large	7" - 8"	11.5 - 1.5 (Youth)	EZBPED-L

ManaMed Inc.
5240 W Charleston Blvd.
Las Vegas NV 89146
United States
Ph: 888-508-0712
www.ManaMed.com



MANAMED™

©2021 ManaMed Inc. All rights reserved.
ManaMed, the ManaMed logo,
EZ Boot Pediatric logo,
are trademarks of ManaMed Inc.
Registered in the U.S.