

*Mana* **E-BOOT**  
**Z** Pediatric

Pediatric Plastic Camboot





# Healing Through Innovation™

The ManaEZ™ Boot Pediatric is a controlled ankle motion walking boot for children and is designed to be worn to help stabilize the foot and ankle after an injury or surgery.



Recommended HCPCS: L4386

## Features & Benefits:


- Rocker bottom to stimulate natural gait.
- Low profile and lightweight.
- Anatomically-shaped plastic uprights to help facilitate a secure fit.
- Easy to apply and adjust with a strapping and liner system.
- Durable and non-slip tread.
- Fits right and left.

## Ideal For:


Mild to severe ankle and foot injuries, fractures, ligament sprains, tendon strains, and postoperative stabilization

Product Name	Child's Foot Length	US Shoe Size	Color
ManaEZ Boot Pediatric Small	Up to 6"	Up to 8.5	Pink
ManaEZ Boot Pediatric Medium	6" - 7"	8.5 - 11.5	Yellow
ManaEZ Boot Pediatric Large	7" - 8"	11.5 - 1.5 (Youth)	Teal

## Fitting Instructions:

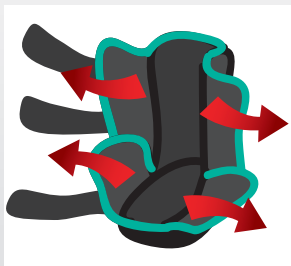

1. Open the boot liner without removing it from the boot.

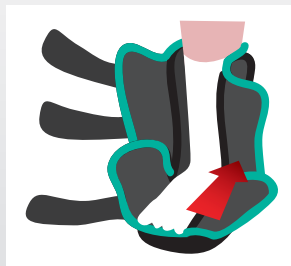

2. Place the boot flat on the floor and put the foot inside the boot. Make sure the heel is all the way back to the back of the boot.



3. Secure the lining to the leg by overlapping the liner on the top of the foot and at the front of the lower leg.



4. Secure the Velcro straps beginning with the straps closest to the toes.

Manufactured By ManaMed™  
5240 W Charleston Blvd.,  
Las Vegas, NV 89146  
888-508-0712 (toll free)  
CustomerService@ManaMed.net  
[www.ManaMed.com](http://www.ManaMed.com)

 **MANAMED™**  
ManaMed and the ManaMed logo are trademarks of ManaMed, Inc.