




# ROYAL BOOT AIRshort


Part Number: RB001S

Suggested HCPC: L4360 / L4361


## INSTRUCTIONS FOR USE

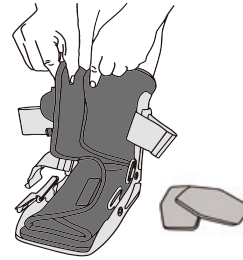


**1:** Place foot into walker boot. Wrap liner around foot and leg. Optional: Foam pads may be placed inside liner or between liner and shell for a better fit or greater comfort.

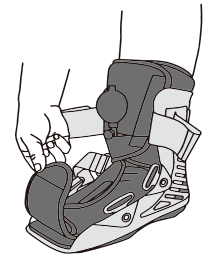


**2:** If using optional toe cover, adjust its positioning if necessary for a proper fit.

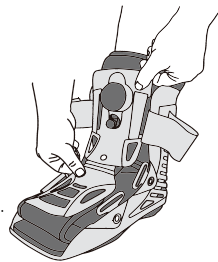

**3:** Place front shell over liner and make sure that the air bubble is aligned with the opening.

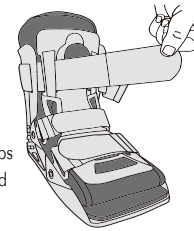



**4:** Thread the straps through the ring and fasten the straps.

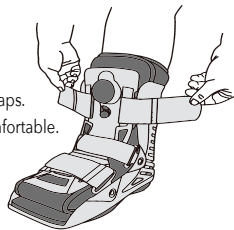

**5:** Fasten foot, ankle and calf straps. Tighten straps until snug and comfortable.



**6:** The Royal Boot Air Short can now be used.

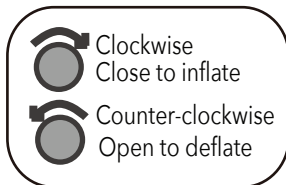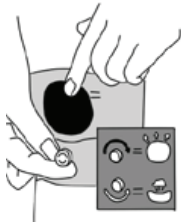

### To Control the Air Pressure:

For additional comfort and a better fit, inflate the air liner. For inflation, close the round valve by turning it clockwise and add air by pressing the air bulb until the desired fit is achieved. For deflation, open the round valve by turning it counter-clockwise and wait 15 seconds until air is released.

**Use and Care:** Hand wash with a mild detergent in luke warm water. Be sure to rinse well since soap residue may cause skin irritation. Air dry.

**Warning:** This device is intended for use only under the advice of a medical professional. If you experience any pain, swelling, sensation changes, or any unusual while using this product, consult your medical professional immediately.

Product Name	Men's Shoe Size	Women's Shoe Size	Part Number
Royal Boot Air Short Extra Small	Up to 5.5	Up to 4	SRB001XS
Royal Boot Air Short Small	6 - 8	4 - 7	SRB001S
Royal Boot Air Short Medium	8 - 11	7 - 10	SRB001M
Royal Boot Air Short Large	11 - 13	10 - 12	SRB001L
Royal Boot Air Short XLarge	13 +	12 +	SRB001XL



**MANAMED™**

ManaMed Inc.  
5240 W Charleston Blvd.  
Las Vegas, NV 89146  
United States  
Ph: 888-508-0712  
www.ManaMed.com

©2019 ManaMed Inc. All rights reserved.  
ManaMed, the ManaMed logo, Royal Boot logo,  
are trademarks of ManaMed Inc.  
Registered in the U.S.