

INSTRUCTIONS FOR USE

Mana **EZ** ANKLE STIRRUP

Size: Universal
Part No: EZAS4350
Sugg. HCPC: **L4350**




Laundrying and Care

1. Remove all panels and/ or any metal components prior to cleaning.
2. This brace can be hand washed in cold or lukewarm water with mild soap or detergent and air dried.
3. Outer shell may be wiped with mild soap and water.
4. Do not use any substance containing chlorine.

Directions:


1. Before application place an cottonsock or ankle wrap on patient's ankle. Spread the shells and place the patient's heel on the heel strap (located between the shells).

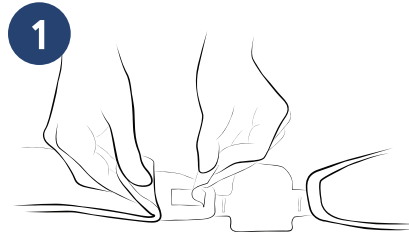


2. Place the shells against the sides of the ankle.



3. Compress the shells around the patient's ankle, while securing straps from bottom to top with velcro closure.

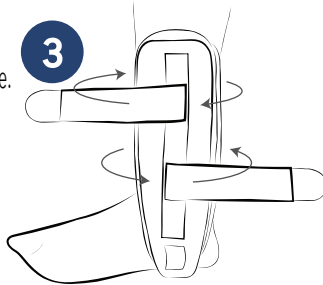

4. Adjust the vertical strap, placing both hands on straps, pulling upward until desired fit. Secure vertical side strap with velcro closure.

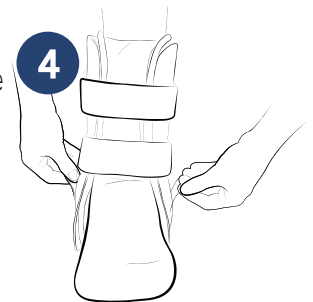

5. Check for adequate circulation. Readjust straps as needed for a secure and comfortable fit.

Warning

1. Carefully read this application instruction and warning prior to fitting braces, failure to properly position the brace and fasten each strap will compromise performance and support.
2. This brace will Not prevent or reduce all injuries. Proper rehabilitation and activity modification are essential while wearing the brace.
3. Federal Law restricts the sale of this Device to anyone other than a Licensed Health Care Provider.

Caution

1. Single patient use only.
2. Do not over-tighten closures.
3. If irritation, pain or discomfort occurs while wearing this brace, discontinue use and immediately consult your medical professional.



MANAMED

ManaMed Inc.
5240 W Charleston Blvd.
Las Vegas NV 89146
United States
Ph: 888-508-0712
www.ManaMed.com

©2020 ManaMed Inc. All rights reserved.
ManaMed, the ManaMed logo,
are trademarks of ManaMed Inc.
Registered in the U.S.