

Indications:

- Applies soft tissue compression around the humerus, controlling its segments.
- Humerus fracture brace relies on gravity for returning one's bone to its normal position.
- Shell of the deltoid brace is a rigid polyethylene that is lightweight and durable.
- The brace is laminated with closed cell foam lining.
- Three straps that secure the rigid components of the brace allow one to maintain this compression as swelling subsides and one's arm muscles diminish.
- Two adjustable straps around the upper arm allow one to customize the fit of the splint for a shoulder/humerus fracture.
- The proximal humerus fracture splint has a strap that wraps beneath the opposite arm, limiting the motion allowed at the shoulder.
- Brace allows some movement, which helps one to make a full recovery.
- Deltoid cap extension of the broken shoulder brace prevents it from migrating.
- 13-inch long upper arm brace.
- Contact closures secure this shoulder brace for a broken shoulder and upper arm.



Fitting instruction

- Open the Velcro fasteners around the upper arm and the pad at the front.
- Place the upper arm cuff around the affected arm and close the main Velcro fastener.
- Take the strap around the back and under the armpit of the uninjured arm to the front and fasten it there. Take care that the fastener for the forearm support sits in the sternum area.
- Close the two straps on the upper arm.
- Place the forearm in the support and fasten with the click closure.

**MANAMED**

ManaMed Inc.
5240 W Charleston Blvd.
Las Vegas NV 89146
United States
Ph: 888-508-0712
www.ManaMed.com

©2020 ManaMed Inc. All rights reserved.
ManaMed, the ManaMed logo,
are trademarks of ManaMed Inc.
Registered in the U.S.