

INSTRUCTIONS FOR USE

ManaEZ SLING

For use with the treatment of:

Surgical repair of rotator cuff.
Anterior dislocation of the G-H joint
Anterior glenohumeral capsular reconstruction
S/P glenohumeral acromioplasty

Part No: EZSLO1



ManaEZ Sling Shoulder Immobilizer with Waist Strap

Size Information

Part Number	Size	Sling Pocket
EZSLO1 S	SM	12.4" x 6.3"
EZSLO1 M	MD	14" x 7"
EZSLO1 L	LG	15.5" x 7.9"
EZSLO1 XL	XL	17" x 8.6"
EZSLO1 XXL	XXL	18.5" x 9.5"



Instructions for use:

1. Place injured arm in sling.
2. Place strap around neck and secure through buckle and adjust for proper height and comfort.
3. Place waist strap around waist and feed strap through D-ring and attach hook and loop closure.

Patients should always consult their physician or health care professional for proper instruction prior to using the EZ Sling Shoulder Immobilizer with Waist Strap.

Cleaning Instructions:

Remove straps before washing
Hand wash in code water
Mild detergent. Air Dry.



ManaMed Inc.
5240 W Charleston Blvd.
Las Vegas NV 89146
United States
Ph: 888-508-0712
www.ManaMed.com

MANAMED

©2020 ManaMed Inc. All rights reserved.
ManaMed, the ManaMed logo,
are trademarks of ManaMed Inc.
Registered in the U.S.